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GB INSTRUCTION FOR USE - Pre-filled Syringe

Intended Use:
The Pre-filled Syringe is intended to be used for foley catheter balloon inflation.

Product Type:
PCAA02, Pre-filled syringe with 10% glycerin, 10ml
PCAA04, Pre-filled syringe with sterile water, 10ml

Warnings and precaution:
Use aseptic technique.
Single use only. Do not reuse.
Not for injection.
Do not use if package is opened or damaged.
Do not use if the cap/adapter on the syringe is not intact.
Do not resterilise.
Sterile unless package opened or damaged.

Contraindications:
No known contraindication.

Storage:
Store in the room with relative humidity less than 80%, good ventilation and no corrosive gas.

Transportation:
Well-packed products can be transported by normal ways, and shall be handled with care during transportation to avoid direct sunlight, violent collision, raining and gravity extrusion.

Side-effects:
None

The combination of the device with product:
To inflate the balloon of urine catheter.

Shelf life:
Three(3) years from end of manufacturing

Disclosure of residual risks:
Not Know

Usage Guidelines:
1. Aseptically open package containing solution prefilled syringe. Visually inspect the contents of each prefilled syringe for clarity, particulate matter, precipitate, discoloration and leakage prior to use. Do not use if any of the aforementioned is observed.
2. Remove and discard syringe cap.
3. Insert luer tip of prefilled syringe into catheter valve with a firm push and twist motion. Pushing the plunger gently until all prefilled liquid inflated into catheter balloon and remove the syringe.
4. Remove and discard unused portions and empty syringes according to the institution's biohazardous waste policy.

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OBELIS S.A Bd Général Wahis 53 B-1030 Brussels Belgium
CMC Medical Devices GmbH Bahnhofstrasse 32 CH-6300 Zug Switzerland