CIDEZYME™

Enzymatic Detergent

- (fr) Détergent enzymatique CIDEZYME™
- (de) CIDEZYME™ -Enzymreiniger
- (it) Detergente enzimatico CIDEZYME™
- (es) Detergente enzimático CIDEZYME™
- (pt) Detergente Enzimático CIDEZYME™
- (bg) Энзимен детергент CIDEZYME™
- (cs) Enzymatický čistící prostředek CIDEZYME™
- (da) CIDEZYME™ enzymatisk rengøringsmiddel
- (el) Ενζυματικό απορρυπαντικό CIDEZYME™
- (et) CIDEZYME™-i ensümaatiline detergent
- (fi) CIDEZYME™-entsyymipesuaine
- (hr) Enzimatski deterdžent CIDEZYME™
- (hu) CIDEZYME™ enzimes tisztítószer
- (id) Detergen Enzimatik CIDEZYME™
- (ja) CIDEZYME™ 酵素洗淨剤
- (lt) CIDEZYME™ fermentinis ploviklis
- (lv) CIDEZYME™ fermentatīvs mazgāšanas līdzeklis
- (mk) CIDEZYME™ ензимски детергент
- (nl) CIDEZYME™ enzymatisch reinigingsmiddel
- (no) CIDEZYME™ enzymatisk vaskemiddel
- (pl) Detergent enzymatyczny CIDEZYME™
- (ro) CIDEZYME™ Detergent enzimatic
- (ru) CIDEZYME™ Ферментное моющее средство
- (sk) Enzymatické čistidlo CIDEZYME™
- (sl) Detergent z encimi CIDEZYME™
- (sv) CIDEZYME™ enzymatiskt rengöringsmedel
- (tr) CIDEZYME™ Enzimatik Deterjan
- (uk) Ферментний м'який засіб CIDEZYME™

ADVANCED STERILIZATION PRODUCTS, INC.

33 Technology Drive, Irvine, CA 92618, USA
 +1(888)-STERRAD  www.asp.com

 ASP, The Netherlands BV
 BIC 1 5657 BX Eindhoven, The Netherlands

English	3
Français	4
Deutsch	5
Italiano	6
Español	7
Português	8
Български	9
Čeština	10
Dansk	11
Ελληνικά	12
Eesti	13
Suomi	14
Hrvatski	15
Magyar	16
Bahasa Indonesia	17
日本語	18
Lietuviškai	19
Latviešu	20
Македонски	21
Nederlands	22
Norsk	23
Polski	24
Română	25
Русский	26
Slovensky	27
Slovenščina	28
Svenska	29
Türkçe	30
Українська	31

CIDEZYME™

Enzymatic Detergent

en

INSTRUCTIONS FOR USE

INDICATIONS FOR USE

CIDEZYME Enzymatic Detergent is intended to be used for pre-soaking and pre-cleaning instruments prior to disinfection or sterilization in accordance with instrument manufacturers' recommendations.

DIRECTIONS FOR USE

1. Add 8 ml of CIDEZYME Enzymatic Detergent liquid concentrate per 1 liter of water. For instruments with dried-on organic matter, use 16 ml per 1 liter of water and/or use warm water.
2. **Soak instruments immediately after use, until all organic matter is dissolved and removed.** A minimum soak of 1 minute is recommended. Extend soak time for instruments with dried-on organic matter. Flush solution through all channels.
3. If necessary, mechanically clean instruments with brushes or soft cloths as recommended by the instrument manufacturer.
4. Thoroughly rinse the instruments, aspirating water through all channels to remove traces of CIDEZYME Detergent.
5. Rough dry instruments, including channels. Instruments are now ready for treatment with a disinfectant.

STORAGE AND DISPOSAL

Store at ambient temperature 15°-30°C (59°-86°F).

Discard used solution after each use.

You may dispose this product in accordance with your National, Federal, State, and Local regulations or per healthcare facility's policies and procedures.

WARNINGS AND PRECAUTIONS

Danger Contains Subtilisin. *May produce an allergic reaction.*

Hazard May cause allergy or asthma symptoms or breathing difficulties if inhaled. The mixture does not meet the criteria for classification.

Precautionary Statement

Prevention Avoid breathing mist. In case of inadequate ventilation wear respiratory protection. Observe good industrial hygiene practices.

Response If inhaled: If breathing is difficult, remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a poison center/doctor. Wash hands after handling.

Storage Store away from incompatible materials.

Disposal Dispose of waste and residues in accordance with local authority requirements.

**For country specific GHS and SDS information, refer to country specific SDS located at asp.com/service-and-support/sds-datasheets.

Do not add bleach or other chemicals. CIDEZYME Detergent is not intended as a solution for holding or storing instruments post cleaning.

HOW SUPPLIED

Reorder Description

2258	1L
2260	5L

For a glossary of symbols used, go to eifu.asp.com

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