




NOBAFLUSH®

Pre-filled Syringe with Saline

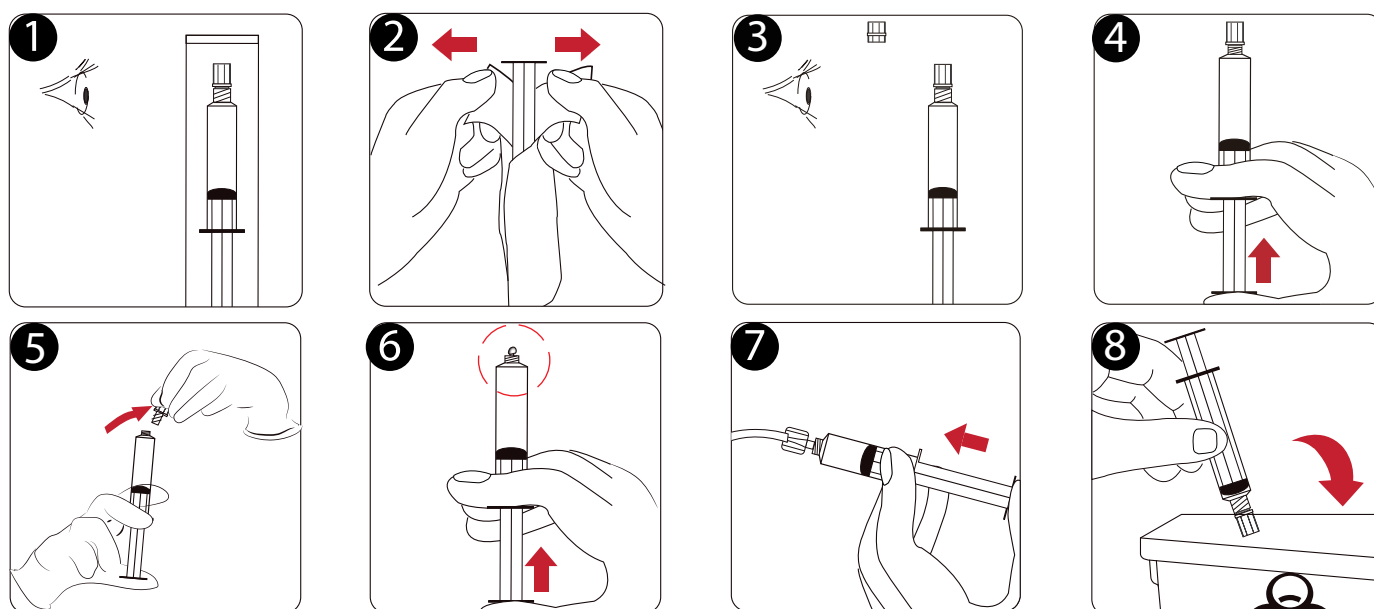

Instructions for Use

1. Visually inspect the package. Do not use if the wrapping is damaged or the solution is leaking.
2. Remove the wrapping.
3. Inspect if the solution is clear, the tip cap is in place, the syringe is intact and there is no evidence of leakage. Do not use if the solution is cloudy or contains particulate matter or if the syringe is damaged or not intact.
4. Depress the plunger with the tip cap on to release the stopper seal.
5. Unscrew the syringe cap from the syringe ensuring not to touch the syringe luer connector to avoid contamination.
6. Hold the syringe upright and push the plunger until the air is expelled from the syringe.
7. Attach with luer connector on injection site and prime to flush with proper force.
8. Discard the empty syringe properly according to local policy. Do not reuse syringes.

Cautions

1. Consult appropriate literature for compatibility data prior to administration and avoid contact between this solution and any incompatible drug products.
2. Do not use if the plastic wrapping is damaged or not intact.
3. Do not use if the product has expired.
4. Do not use if the syringe tip cap or stopper is damaged in any way.
5. Do not use if there is any indication of syringe leakage.
6. Do not use if the solution contains a precipitate or has any type of suspended particulate matter.
7. Do not reuse. For single use only. Promptly discard partially used products in accordance with your facilities' policies.
8. Some patients may experience common symptoms of temporary cold sensation or unpleasant taste or smell after administration. A slower administration push rate may help to reduce this from occurring.
9. Improper use can lead to adverse effects such as air embolism. Consult and follow directions for use prior to administration.

NOTIFICATION for EU Customer: Any serious incident that has occurred in relation to the device must be reported to the manufacturer and distributor and the competent authority of the EU Member State where the Customer's facility is established.

STERILE

Sterile

STERILE R

Sterilized using irradiation



Consult instructions for use



Store at controlled temperature 15-25 °C

CE 0197



Not made with Natural Rubber Latex



Not made with DEHP



Not made with BPA



Keep away from sunlight



Keep Dry



Do not use if package is damaged



Do not resterilize



Caution



Medical Device



Single use only

Rx ONLY

EU REP Shanghai International Holding Corp.GmbH(Europe) Eiffestrasse 80,20537 Hamburg Germany

Jiangsu Caina Medical Co. Ltd, No.253 Chenglu Road, Huashi Town, Jiangyin City, Jiangsu, China 214421

NOBAMED Paul Danz AG, Höltenstr. 1-5, D-58300 Wetter (Ruhr), Germany